

EZ PlusTM

Internal



EZ Plus™ Internal

Get the Maximum Benefits

Submerged Type Implant, Trip Internal Connection

Design The EZ Plus fixture's crestal tapered design provides maximum initial stability in any bone quality. In dense bone situations, the self tapping thread design and four cutting edges of our fixture provide easy insertion. The thread design that runs to the top of the fixture has been engineered to maximize the connection between the bone and the implant, while minimizing crestal bone loss. Platform switching enhances crestal bone attachment.

Surface Our Super RBM Surface Treatment produces a favorable surface topography that has been proven through extensive systematic testing. MegaGen's Super RBM Surface shortens loading time and provides favorable results in any bone quality.

Feature Internal Trip Connection : MegaGen's Trip anti-rotation structure will not deform until 220N of insertion torque, allowing for extremely stable initial fixation.

Surgery On a case by case basis, proper initial fixation can be achieved using the EZ Plus Counter Sink Drill. The mount-free type handpiece connector with a built-in ceramic ball provides stability so that fixture adhesion can be maintained even after many repeated uses.

Prosthetics The EZ Plus system incorporates an 11° morse taper, producing a conical seal which forms a cold weld between the abutment and the implant. Abutment options to provide esthetic and prosthetic excellence include UCLA, Angled, Solid, Post, Milling type, Ceramic, and O-Ring abutments as well as various other abutment options.


Packaging

Pre-Mount Type : Fixture Connected with Multi Post and Cover Screw [MB]

No-Mount Type : Fixture and Cover Screw [MF]

	Small	Regular			Wide		
Type	MF	MB	MF	MB	MF	MB	MF
Thread Diameter	Ø3.3	Ø4.0		Ø4.5		Ø5.0	
Body Length		7.0 / 8.5 / 10 / 11.5 / 13 / 15mm					

Strong Internal Trip™

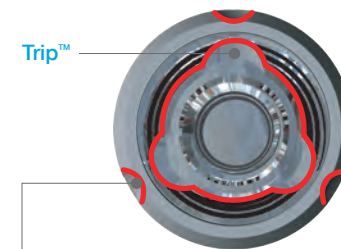


Trip™ Indication Groove

Three grooves on fixture crestal bevel area locate same direction with Internal Trip of EZ Plus implant. Connection to handpiece is easy for delivering implant to patient oral.

Convenience of Mount Free



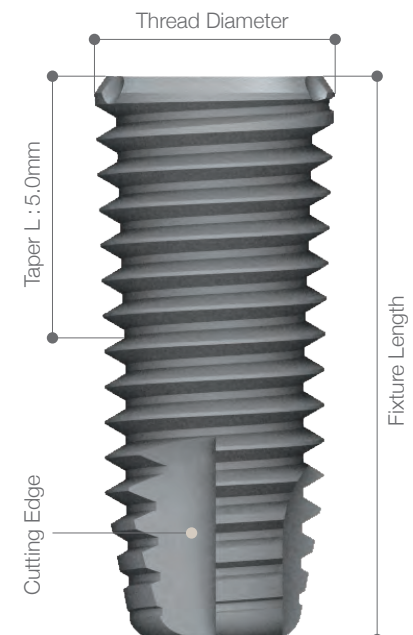




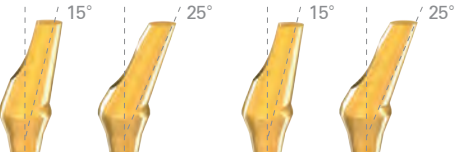







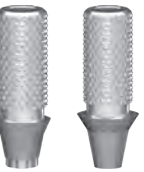













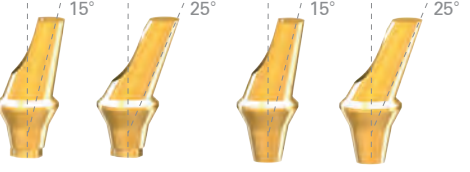


EZ Plus MF is a Mount Free type implant. Free from inconvenience to remove the mount after placing the implant. Enjoy the convenience of Mount Free & Economic Price.



EZ Plus™ Internal / Abutments

Dental Implant Specialty Company, **MegaGen!**

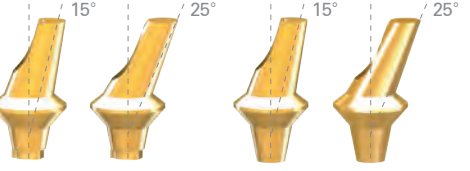■ Small
 ■ Regular & Wide
 Ref: Reference Code
 ● Non-Octagonal Type
 ● Octagonal Type
 ● Trip Type
 CH: Cuff Height
 PD: Profile Diameter
 PH: Post Height

Cover Screw	Healing Abutment	Impression Coping [Transfer Type]	Abutment Screw	Temporary Abutment	Gold Abutment	Multi Post	Angled Abutment	Octa Abutment	Impression Coping [Pick-up Type]	Gold Abutment
1.2 Hex ●  Ref ■ EPCS100	PD: Ø4.0 CH: 3/5/7mm 1.2 Hex ● 		 Ø1.6			Multi Post Screw 1.2 Hex ●  EZIMS200S	PD: Ø4.0 CH: 2/4mm 	PD: Ø4.8 CH: 2/3/4/5mm 	 Guide Pin 	
	■ EPHA403 EPHA405 EPHA407	■ EPIT100 ■ EPIT200T EPIG00S	■ EPAS100	■ EPTH200T EPTN100T	■ EPGA200T EPGA100T		■ EPAH4215T EPAH4415T EPAH4225T EPAH4425T EPAA4215T EPAA4415T EPAA4225T EPAA4425T	■ EPOAS501 EPOAS502 EPOAS503 EPOAS504 EPOAS505	RIOG100 RIOG150 RIOH100T RION100T	IOGO100T IOGN100T
	PD: Ø5.0 CH: 3/5/7mm 1.2 Hex ● 		 Ø2.0			PD: Ø5.0 CH: 2mm  EZIMB500T	PD: 5.0 CH: 2/3/4/5 mm 	PD: Ø4.8 CH: 2/3/4,5mm 	 Guide Pin 	
	■ AMHA503 AMHA505 AMHA507	■ REIN500I ■ REIH500IT EPIG00R	■ AMAS100	■ EZITH100T EZITN100T	■ AMGA200T AMGA100T		■ AEZA5215T AEZA5315T AEZA5415T AEZA5515T AEZA5225T AEZA5325T AEZA5425T AEZA5525T AMAA5215T AMAA5315T AMAA5415T AMAA5515T AMAA5225T AMAA5325T AMAA5425T AMAA5525T	■ EPOAR501 EPOAR502 EPOAR503 EPOAR504 EPOAR505	IOA300	IOPH100T IOPN100T
	PD: Ø6.0 CH: 3/5/7mm 1.2 Hex ● 	 Guide Pin  Guide Pin				PD: Ø6.0 CH: 2mm  EZIMB600T	PD: Ø6.0 CH: 2/3/4/5 mm 		 1.2 Hex ●	Compatible with ExFeel Internal Octa Abutments
■ EPCS200	■ AMHA603 AMHA605 AMHA607	■ EPIPS200T EPIG05S ■ EPIPR200T EPIG10R EPIPR100T	■ EILA330	■ EILA400			■ AEZA6215T AEZA6315T AEZA6415T AEZA6515T AEZA6225T AEZA6325T AEZA6425T AEZA6525T AMAA6215T AMAA6315T AMAA6415T AMAA6515T AMAA6225T AMAA6325T AMAA6425T AMAA6525T	IHC400T	IRCS200	





EZ Plus™ Internal / Abutments

Dental Implant Specialty Company, **MegaGen!**

■ Small
 ■ Regular & Wide
 Ref: Reference Code
 ● Non-Octagonal Type
 ● Octagonal Type
 Trip Type
 CH: Cuff Height
 PD: Profile Diameter
 PH: Post Height

EZ Post	Comfort Cap	Snap Impression Coping	Lab Analog
PD: Ø4.0 CH: 1/2/3/4/5mm 	PD: Ø4.0 	PD: Ø4.0 	
Ref: ■ EPEH 4155T EPEH 4155T EPEH 4255T EPEH 4255T EPEH 4355T EPEH 4355T EPEH 4455T EPEH 4455T EPEH 4555T EPEH 4555T	■ AMIC 04	■ AMIIC 04	■ AMSLA 450
PD: Ø5.0 CH: 1/2/3/4/5mm 	PD: Ø5.0 	PD: Ø5.0 	
■ AEZP 5150T AMMP 5150T AEZP 5250T AMMP 5250T AEZP 5350T AMMP 5350T AEZP 5450T AMMP 5450T AEZP 5550T AMMP 5550T	■ AMIC 05	■ AMIIC 05	■ AMSLA 550
PD: Ø6.0 CH: 1/2/3/4/5mm 	PD: Ø6.0 	PD: Ø6.0 	
■ AEZP 6150T AMMP 6150T AEZP 6250T AMMP 6250T AEZP 6350T AMMP 6350T AEZP 6450T AMMP 6450T AEZP 6550T AMMP 6550T	■ AMIC 06	■ AMIIC 06	■ AMSLA 650

Solid Abutment	Comfort Cap	Snap Impression Coping	Lab Analog
PD: Ø4.0 CH: 1/2/3/4/5mm 	PD: Ø4.0 	PD: Ø4.0 	
■ EPS 4155 EPS 4155 EPS 4255 EPS 4255 EPS 4355 EPS 4355 EPS 4455 EPS 4455 EPS 4555 EPS 4555	■ AMIC 04	■ AMIIC 04	■ AMSLA 450
PD: Ø5.0 CH: 1/2/3/4/5mm 	PD: Ø5.0 	PD: Ø5.0 	
■ AMMS 5135 AMMS 5150 AMMS 5235 AMMS 5250 AMMS 5335 AMMS 5350 AMMS 5435 AMMS 5450 AMMS 5535 AMMS 5550	■ AMIC 05	■ AMIIC 05	■ AMSLA535 AMSLA 550
PD: Ø6.0 CH: 1/2/3/4/5mm 	PD: Ø6.0 	PD: Ø6.0 	
■ AMMS 6135 AMMS 6150 AMMS 6235 AMMS 6250 AMMS 6335 AMMS 6350 AMMS 6435 AMMS 6450 AMMS 6535 AMMS 6550	■ AMIC 06	■ AMIIC 06	■ AMSLA 635 AMSLA 650

Milling Abutment	
PD: Ø5.0 CH: 1/2/3/4/5mm 	PD: Ø5.0 CH: 1/2/3/4/5mm 
■ AEZP 5190T AMMP 5190T AEZP 5290T AMMP 5290T AEZP 5390T AMMP 5390T AEZP 5490T AMMP 5490T AEZP 5590T AMMP 5590T	■ AMMS 5190 AMMS 5290 AMMS 5390 AMMS 5490 AMMS 5590
PD: Ø6.0 CH: 1/2/3/4/5mm 	PD: Ø6.0 CH: 1/2/3/4/5mm 
■ AEZP 6190T AMMP 6190T AEZP 6290T AMMP 6290T AEZP 6390T AMMP 6390T AEZP 6490T AMMP 6490T AEZP 6590T AMMP 6590T	■ AMMS 6190 AMMS 6290 AMMS 6390 AMMS 6490 AMMS 6590



EZ Plus™ Internal

Dental Implant Specialty Company, **MegaGen!**

■ EZ Plus Internal Fixture

Fixture [Pre-Mount]		
Ref. Code	Description	Page
EZIR4007P	Regular / 4.0 x 7.0mm / MB	p1
EZIR4008P	Regular / 4.0 x 8.5mm / MB	
EZIR4010P	Regular / 4.0 x 10.0mm / MB	
EZIR4011P	Regular / 4.0 x 11.5mm / MB	
EZIR4013P	Regular / 4.0 x 13.0mm / MB	
EZIR4015P	Regular / 4.0 x 15.0mm / MB	
EZIR4507P	Regular / 4.5 x 7.0mm / MB	p1
EZIR4508P	Regular / 4.5 x 8.5mm / MB	
EZIR4510P	Regular / 4.5 x 10.0mm / MB	
EZIR4511P	Regular / 4.5 x 11.5mm / MB	
EZIR4513P	Regular / 4.5 x 13.0mm / MB	
EZIR4515P	Regular / 4.5 x 15.0mm / MB	
EZIW5007P	Wide / 5.0 x 7.0mm / MB	p1
EZIW5008P	Wide / 5.0 x 8.5mm / MB	
EZIW5010P	Wide / 5.0 x 10.0mm / MB	
EZIW5011P	Wide / 5.0 x 11.5mm / MB	
EZIW5013P	Wide / 5.0 x 13.0mm / MB	
EZIW5015P	Wide / 5.0 x 15.0mm / MB	
Fixture [No-Mount]		
EZIS3307C	Small / 3.3 x 7.0mm / MF	p1
EZIS3308C	Small / 3.3 x 8.5mm / MF	
EZIS3310C	Small / 3.3 x 10.0mm / MF	
EZIS3311C	Small / 3.3 x 11.5mm / MF	
EZIS3313C	Small / 3.3 x 13.0mm / MF	
EZIS3315C	Small / 3.3 x 15.0mm / MF	
EZIR4007C	Regular / 4.0 x 7.0mm / MF	p1
EZIR4008C	Regular / 4.0 x 8.5mm / MF	
EZIR4010C	Regular / 4.0 x 10.0mm / MF	
EZIR4011C	Regular / 4.0 x 11.5mm / MF	
EZIR4013C	Regular / 4.0 x 13.0mm / MF	
EZIR4015C	Regular / 4.0 x 15.0mm / MF	
EZIR4507C	Regular / 4.5 x 7.0mm / MF	p1
EZIR4508C	Regular / 4.5 x 8.5mm / MF	
EZIR4510C	Regular / 4.5 x 10.0mm / MF	
EZIR4511C	Regular / 4.5 x 11.5mm / MF	
EZIR4513C	Regular / 4.5 x 13.0mm / MF	
EZIR4515C	Regular / 4.5 x 15.0mm / MF	
EZIW5007C	Wide / 5.0 x 7.0mm / MF	p1
EZIW5008C	Wide / 5.0 x 8.5mm / MF	
EZIW5010C	Wide / 5.0 x 10.0mm / MF	
EZIW5011C	Wide / 5.0 x 11.5mm / MF	
EZIW5013C	Wide / 5.0 x 13.0mm / MF	
EZIW5015C	Wide / 5.0 x 15.0mm / MF	

■ Instrument

Instrument		
Ref. Code	Description	Page
MGD100L	Guide Drill	p2
TEEL200M	Lindermann Drill	
TEED2007	Twist Drill / Ø2.0 Stopper / 7mm	p2
TEED2008	Twist Drill / Ø2.0 Stopper / 8.5mm	
TEED2010	Twist Drill / Ø2.0 Stopper / 10mm	
TEED2011	Twist Drill / Ø2.0 Stopper / 11.5mm	
TEED2807	Shaping Drill / Ø2.8 Stopper / 7mm	p2
TEED2808	Shaping Drill / Ø2.8 Stopper / 8.5mm	
TEED2810	Shaping Drill / Ø2.8 Stopper / 10mm	
TEED2811	Shaping Drill / Ø2.8 Stopper / 11.5mm	
TEED200M	Twist Drill / Ø2.0mm	p2
TEED280M	Straight Drill / Ø2.8mm	
TEED330M	Straight Drill / Ø3.3mm	
TEED380M	Straight Drill / Ø3.8mm	
TEED430M	Straight Drill / Ø4.3mm	
MPD2833L	Pilot Drill / Ø2.0 → Ø2.8 → Ø3.3mm	p2
MPD3843L	Pilot Drill / Ø3.3 → Ø3.8 → Ø4.3mm	
MCS330N	Countersink / Ø3.3mm	p2
MCS400N	Countersink / Ø4.0mm	
MCS450N	Countersink / Ø4.5mm	
MCS500N	Countersink / Ø5.0mm	
MTP330L	Tap / Ø3.3mm	p2
MTP400L	Tap / Ø4.0mm	
MTP450L	Tap / Ø4.5mm	
MTP500S	Tap / Ø5.0mm	
MDE150	Drill Extension	p2
MHD090L	Hand Driver	
MHD120L	Hand Driver	
MDI100	Direction Indicator	
MRE400S	Ratchet Extension Plus	
MEIHC330L	Handpiece Connector / Small / Long	p2
MEIHC400S	Handpiece Connector / Regular & Wide / Short	
MEIHC400L	Handpiece Connector / Regular & Wide / Long	
MEIRE330L	Ratchet Extension / Small / Long	p2
MEIRE400S	Ratchet Extension / Regular & Wide / Short	
MEIRE400L	Ratchet Extension / Regular & Wide / Long	
MRW040S	Ratchet Wrench	p2

Fixture

Instrument

Abutment

Ref. Code

Memo



EZ Plus™ Internal

Dental Implant Specialty Company, **MegaGen!**

CH : Cuff Height PD : Profile Diameter PH : Post Height SH : Solid Height NT : Non-Trip TP : Trip

■ Abutment

Cover Screw

Ref. Code	Description	Page
EPCS100	Cover Screw / Small	p3
EPCS200	Cover Screw / Regular & Wide	

Healing Abutment

EPHA403	PD=∅4.0 / CH=3mm	p3
EPHA405	PD=∅4.0 / CH=5mm	
EPHA407	PD=∅4.0 / CH=7mm	
AMHA503	PD=∅5.0 / CH=3mm	p3
AMHA505	PD=∅5.0 / CH=5mm	
AMHA507	PD=∅5.0 / CH=7mm	
AMHA603	PD=∅6.0 / CH=3mm	p3
AMHA605	PD=∅6.0 / CH=5mm	
AMHA607	PD=∅6.0 / CH=7mm	

Impression Coping & Guide Pin

EPIT100	Impression Coping / Small Transfer NT	p3
EPIT200T	Impression Coping / Small Transfer TP	
EPIPS100T	Impression Coping / Small Pick-up NT	
EPIPS200T	Impression Coping / Small Pick-up TP	
REIN500I	Impression Coping / Regular & Wide Transfer NT	p3
REIH500IT	Impression Coping / Regular & Wide Transfer TP	
EPIPR100T	Impression Coping / Regular & Wide Pick-up NT	
EPIPR200T	Impression Coping / Regular & Wide Pick-up TP	
EPIG00S	Guide Pin / Small / EPIT200	p3
EPIG05S	Guide Pin / Small / EPIPS200	
EPIG10S	Guide Pin / Small / EPIPS200	
EPIG00R	Guide Pin / Regular & Wide / REIH500I	p3
EPIG05R	Guide Pin / Regular & Wide / EPIPR200	
EPIG10R	Guide Pin / Regular & Wide / EPIPR200	
AMIIC04	Snap Impression Coping / PD=∅4.0mm	p3
AMIIC05	Snap Impression Coping / PD=∅5.0mm	
AMIIC06	Snap Impression Coping / PD=∅6.0mm	

Temporary Abutment

EPTN100T	Small NT	p3
EPTH200T	Small TP	
EZITN100T	Regular & Wide NT	p3
EZITH100T	Regular & Wide TP	

Lab Analog

EILA330	Fixture Lv / Small	p3
EILA400	Fixture Lv / Regular & Wide	

Solid & Post Lab Analog

AMSLA450	Solid Lv / Small / PD=4.0 / SH=5.5mm	p3
AMSLA535	Solid Lv / Regular & Wide / PD=5.0 / SH=4.0mm	
AMSLA550	Solid Lv / Regular & Wide / PD=5.0 / SH=5.5mm	
AMSLA635	Solid Lv / Regular & Wide / PD=6.0 / SH=4.0mm	
AMSLA650	Solid Lv / Regular & Wide / PD=6.0 / SH=5.5mm	

Multi Post

Ref. Code	Description	Page
EZIMB500T	Multi Post / Regular	p3
EZIMB600T	Multi Post / Wide	
EZIMS200	Multi Post Screw	

Abutment Screw

EPAS100	Abutment Screw / Small	p3
AMAS100	Abutment Screw / Regular & Wide	

Angled Abutment - 15° [Non-Trip]

EPAA4215T	Small / PD=∅4.0 / CH=2mm / PH=7mm	p3
EPAA4415T	Small / PD=∅4.0 / CH=4mm / PH=7mm	
AMAA5215T	Regular & Wide / PD=∅5.0 / CH=2mm / PH=7mm	p3
AMAA5315T	Regular & Wide / PD=∅5.0 / CH=3mm / PH=7mm	
AMAA5415T	Regular & Wide / PD=∅5.0 / CH=4mm / PH=7mm	
AMAA5515T	Regular & Wide / PD=∅5.0 / CH=5mm / PH=7mm	
AMAA6215T	Regular & Wide / PD=∅6.0 / CH=2mm / PH=7mm	
AMAA6315T	Regular & Wide / PD=∅6.0 / CH=3mm / PH=7mm	
AMAA6415T	Regular & Wide / PD=∅6.0 / CH=4mm / PH=7mm	
AMAA6515T	Regular & Wide / PD=∅6.0 / CH=5mm / PH=7mm	

Angled Abutment - 25° [Non-Trip]

EPAA4225T	Small / PD=∅4.0 / CH=2mm / PH=7mm	p3
EPAA4425T	Small / PD=∅4.0 / CH=4mm / PH=7mm	
AMAA5225T	Regular & Wide / PD=∅5.0 / CH=2mm / PH=7mm	p3
AMAA5325T	Regular & Wide / PD=∅5.0 / CH=3mm / PH=7mm	
AMAA5425T	Regular & Wide / PD=∅5.0 / CH=4mm / PH=7mm	
AMAA5525T	Regular & Wide / PD=∅5.0 / CH=5mm / PH=7mm	
AMAA6225T	Regular & Wide / PD=∅6.0 / CH=2mm / PH=7mm	
AMAA6325T	Regular & Wide / PD=∅6.0 / CH=3mm / PH=7mm	
AMAA6425T	Regular & Wide / PD=∅6.0 / CH=4mm / PH=7mm	
AMAA6525T	Regular & Wide / PD=∅6.0 / CH=5mm / PH=7mm	

Angled Abutment - 15° [Trip]

EPAH4215T	Small / PD=∅4.0 / CH=2mm / PH=7mm	p3
EPAH4415T	Small / PD=∅4.0 / CH=4mm / PH=7mm	
AEZA5215T	Regular & Wide / PD=∅5.0 / CH=2mm / PH=7mm	p3
AEZA5315T	Regular & Wide / PD=∅5.0 / CH=3mm / PH=7mm	
AEZA5415T	Regular & Wide / PD=∅5.0 / CH=4mm / PH=7mm	
AEZA5515T	Regular & Wide / PD=∅5.0 / CH=5mm / PH=7mm	
AEZA6215T	Regular & Wide / PD=∅6.0 / CH=2mm / PH=7mm	
AEZA6315T	Regular & Wide / PD=∅6.0 / CH=3mm / PH=7mm	
AEZA6415T	Regular & Wide / PD=∅6.0 / CH=4mm / PH=7mm	
AEZA6515T	Regular & Wide / PD=∅6.0 / CH=5mm / PH=7mm	



EZ Plus™ Internal

Dental Implant Specialty Company, MegaGen!

CH : Cuff Height PD : Profile Diameter PH : Post Height NT : Non-Trip TP : Trip

Angled Abutment - 25° [Trip]

Ref. Code	Description	Page
EPAH4225T	Small / PD=∅4.0 / CH=2mm / PH=7mm	p3
EPAH4425T	Small / PD=∅4.0 / CH=4mm / PH=7mm	
AEZA5225T	Regular & Wide / PD=∅5.0 / CH=2mm / PH=7mm	p3
AEZA5325T	Regular & Wide / PD=∅5.0 / CH=3mm / PH=7mm	
AEZA5425T	Regular & Wide / PD=∅5.0 / CH=4mm / PH=7mm	
AEZA5525T	Regular & Wide / PD=∅5.0 / CH=5mm / PH=7mm	
AEZA6225T	Regular & Wide / PD=∅6.0 / CH=2mm / PH=7mm	
AEZA6325T	Regular & Wide / PD=∅6.0 / CH=3mm / PH=7mm	
AEZA6425T	Regular & Wide / PD=∅6.0 / CH=4mm / PH=7mm	
AEZA6525T	Regular & Wide / PD=∅6.0 / CH=5mm / PH=7mm	

Octa Abutment

EPOAS501	Small / CH=1mm	p3
EPOAS502	Small / CH=2mm	
EPOAS503	Small / CH=3mm	
EPOAS504	Small / CH=4mm	
EPOAS505	Small / CH=5mm	
EPOAR501	Regular & Wide / CH=1mm	p3
EPOAR502	Regular & Wide / CH=2mm	
EPOAR503	Regular & Wide / CH=3mm	
EPOAR504	Regular & Wide / CH=4mm	
EPOAR505	Regular & Wide / CH=5mm	

Octa Healing Cap

IHC400T	Healing Cap	p3
---------	-------------	----

Octa Impression Coping

RION100T	Impression Coping / Non-Octa	p3
RIOH100T	Impression Coping / Octa	
RIOG100	Guide Pin / 10mm	
RIOG150	Guide Pin / 15mm	

Octa Lab Analog

IOA300	Octa Lab Analog	p3
--------	-----------------	----

Octa Gold Abutment

IOGN100T	Gold Abutment / Non-Octa	p3
IOGO100T	Gold Abutment / Octa	

Octa Plastic Abutment

IOPN100T	Plastic Abutment / Non-Octa	p3
IOPH100T	Plastic Abutment / Octa	

Gold Abutment

EPGA100T	Small NT	p3
EPGA200T	Small TP	
AMGA100T	Regular & Wide NT	p3
AMGA200T	Regular & Wide TP	

EZ Post [Non-Trip]

Ref. Code	Description	Page
EPEN4155T	Small / PD=∅4.0 / CH=1mm / PH=5.5mm	p4
EPEN4255T	Small / PD=∅4.0 / CH=2mm / PH=5.5mm	
EPEN4355T	Small / PD=∅4.0 / CH=3mm / PH=5.5mm	
EPEN4455T	Small / PD=∅4.0 / CH=4mm / PH=5.5mm	
EPEN4555T	Small / PD=∅4.0 / CH=5mm / PH=5.5mm	
AMMP5150T	Regular & Wide / PD=∅5.0 / CH=1mm / PH=5.0mm	p4
AMMP5250T	Regular & Wide / PD=∅5.0 / CH=2mm / PH=5.0mm	
AMMP5350T	Regular & Wide / PD=∅5.0 / CH=3mm / PH=5.0mm	
AMMP5450T	Regular & Wide / PD=∅5.0 / CH=4mm / PH=5.0mm	
AMMP5550T	Regular & Wide / PD=∅5.0 / CH=5mm / PH=5.0mm	
AMMP6150T	Regular & Wide / PD=∅6.0 / CH=1mm / PH=5.0mm	
AMMP6250T	Regular & Wide / PD=∅6.0 / CH=2mm / PH=5.0mm	
AMMP6350T	Regular & Wide / PD=∅6.0 / CH=3mm / PH=5.0mm	
AMMP6450T	Regular & Wide / PD=∅6.0 / CH=4mm / PH=5.0mm	
AMMP6550T	Regular & Wide / PD=∅6.0 / CH=5mm / PH=5.0mm	

Milling Abutment [Non-Trip]

AMMP5190T	Regular & Wide / PD=∅5.0 / CH=1mm / PH=9.0mm	p4
AMMP5290T	Regular & Wide / PD=∅5.0 / CH=2mm / PH=9.0mm	
AMMP5390T	Regular & Wide / PD=∅5.0 / CH=3mm / PH=9.0mm	
AMMP5490T	Regular & Wide / PD=∅5.0 / CH=4mm / PH=9.0mm	
AMMP5590T	Regular & Wide / PD=∅5.0 / CH=5mm / PH=9.0mm	
AMMP6190T	Regular & Wide / PD=∅6.0 / CH=1mm / PH=9.0mm	
AMMP6290T	Regular & Wide / PD=∅6.0 / CH=2mm / PH=9.0mm	
AMMP6390T	Regular & Wide / PD=∅6.0 / CH=3mm / PH=9.0mm	
AMMP6490T	Regular & Wide / PD=∅6.0 / CH=4mm / PH=9.0mm	
AMMP6590T	Regular & Wide / PD=∅6.0 / CH=5mm / PH=9.0mm	

EZ Post [Trip]

EPEH4155T	Small / PD=∅4.0 / CH=1mm / PH=5.5mm	p4
EPEH4255T	Small / PD=∅4.0 / CH=2mm / PH=5.5mm	
EPEH4355T	Small / PD=∅4.0 / CH=3mm / PH=5.5mm	
EPEH4455T	Small / PD=∅4.0 / CH=4mm / PH=5.5mm	
EPEH4555T	Small / PD=∅4.0 / CH=5mm / PH=5.5mm	
AEZP5150T	Regular & Wide / PD=∅5.0 / CH=1mm / PH=5.0mm	p4
AEZP5250T	Regular & Wide / PD=∅5.0 / CH=2mm / PH=5.0mm	
AEZP5350T	Regular & Wide / PD=∅5.0 / CH=3mm / PH=5.0mm	
AEZP5450T	Regular & Wide / PD=∅5.0 / CH=4mm / PH=5.0mm	
AEZP5550T	Regular & Wide / PD=∅5.0 / CH=5mm / PH=5.0mm	
AEZP6150T	Regular & Wide / PD=∅6.0 / CH=1mm / PH=5.0mm	
AEZP6250T	Regular & Wide / PD=∅6.0 / CH=2mm / PH=5.0mm	
AEZP6350T	Regular & Wide / PD=∅6.0 / CH=3mm / PH=5.0mm	
AEZP6450T	Regular & Wide / PD=∅6.0 / CH=4mm / PH=5.0mm	
AEZP6550T	Regular & Wide / PD=∅6.0 / CH=5mm / PH=5.0mm	

Fixture

Instrument

Abutment

Ref. Code

Memo



EZ PlusTM Internal

Dental Implant Specialty Company, **MegaGen!**

CH : Cuff Height PD : Profile Diameter SH : Solid Height

Milling Abutment [Trip]

Ref. Code	Description	Page
AEZP5190T	Regular & Wide / PD=Ø5.0 / CH=1mm / PH=9.0mm	p4
AEZP5290T	Regular & Wide / PD=Ø5.0 / CH=2mm / PH=9.0mm	
AEZP5390T	Regular & Wide / PD=Ø5.0 / CH=3mm / PH=9.0mm	
AEZP5490T	Regular & Wide / PD=Ø5.0 / CH=4mm / PH=9.0mm	
AEZP5590T	Regular & Wide / PD=Ø5.0 / CH=5mm / PH=9.0mm	
AEZP6190T	Regular & Wide / PD=Ø6.0 / CH=1mm / PH=9.0mm	
AEZP6290T	Regular & Wide / PD=Ø6.0 / CH=2mm / PH=9.0mm	
AEZP6390T	Regular & Wide / PD=Ø6.0 / CH=3mm / PH=9.0mm	
AEZP6490T	Regular & Wide / PD=Ø6.0 / CH=4mm / PH=9.0mm	
AEZP6590T	Regular & Wide / PD=Ø6.0 / CH=5mm / PH=9.0mm	

Solid Abutment

EPS4155	Small / PD=Ø4.0 / CH=1mm / SH=5.5mm	p4
EPS4255	Small / PD=Ø4.0 / CH=2mm / SH=5.5mm	
EPS4355	Small / PD=Ø4.0 / CH=3mm / SH=5.5mm	
EPS4455	Small / PD=Ø4.0 / CH=4mm / SH=5.5mm	
EPS4555	Small / PD=Ø4.0 / CH=5mm / SH=5.5mm	
AMMS5135	Regular & Wide / PD=Ø5.0 / CH=1mm / SH=3.5mm	p4
AMMS5235	Regular & Wide / PD=Ø5.0 / CH=2mm / SH=3.5mm	
AMMS5335	Regular & Wide / PD=Ø5.0 / CH=3mm / SH=3.5mm	
AMMS5435	Regular & Wide / PD=Ø5.0 / CH=4mm / SH=3.5mm	
AMMS5535	Regular & Wide / PD=Ø5.0 / CH=5mm / SH=3.5mm	
AMMS5150	Regular & Wide / PD=Ø5.0 / CH=1mm / SH=5.0mm	
AMMS5250	Regular & Wide / PD=Ø5.0 / CH=2mm / SH=5.0mm	
AMMS5350	Regular & Wide / PD=Ø5.0 / CH=3mm / SH=5.0mm	
AMMS5450	Regular & Wide / PD=Ø5.0 / CH=4mm / SH=5.0mm	
AMMS5550	Regular & Wide / PD=Ø5.0 / CH=5mm / SH=5.0mm	
AMMS6135	Regular & Wide / PD=Ø6.0 / CH=1mm / SH=3.5mm	p4
AMMS6235	Regular & Wide / PD=Ø6.0 / CH=2mm / SH=3.5mm	
AMMS6335	Regular & Wide / PD=Ø6.0 / CH=3mm / SH=3.5mm	
AMMS6435	Regular & Wide / PD=Ø6.0 / CH=4mm / SH=3.5mm	
AMMS6535	Regular & Wide / PD=Ø6.0 / CH=5mm / SH=3.5mm	
AMMS6150	Regular & Wide / PD=Ø6.0 / CH=1mm / SH=5.0mm	
AMMS6250	Regular & Wide / PD=Ø6.0 / CH=2mm / SH=5.0mm	
AMMS6350	Regular & Wide / PD=Ø6.0 / CH=3mm / SH=5.0mm	
AMMS6450	Regular & Wide / PD=Ø6.0 / CH=4mm / SH=5.0mm	
AMMS6550	Regular & Wide / PD=Ø6.0 / CH=5mm / SH=5.0mm	

Milling Abutment

AMMS5190	Regular & Wide / PD=Ø5.0 / CH=1mm / SH=9.0mm	p4
AMMS5290	Regular & Wide / PD=Ø5.0 / CH=2mm / SH=9.0mm	
AMMS5390	Regular & Wide / PD=Ø5.0 / CH=3mm / SH=9.0mm	
AMMS5490	Regular & Wide / PD=Ø5.0 / CH=4mm / SH=9.0mm	
AMMS5590	Regular & Wide / PD=Ø5.0 / CH=5mm / SH=9.0mm	
AMMS6190	Regular & Wide / PD=Ø6.0 / CH=1mm / SH=9.0mm	
AMMS6290	Regular & Wide / PD=Ø6.0 / CH=2mm / SH=9.0mm	
AMMS6390	Regular & Wide / PD=Ø6.0 / CH=3mm / SH=9.0mm	
AMMS6490	Regular & Wide / PD=Ø6.0 / CH=4mm / SH=9.0mm	
AMMS6590	Regular & Wide / PD=Ø6.0 / CH=5mm / SH=9.0mm	

Comfort Cap

Ref. Code	Description	Page
AMIC04	Comfort Cap for Ø4.0	p4
AMIC05	Comfort Cap for Ø5.0	
AMIC06	Comfort Cap for Ø6.0	

Snap Impression Coping

AMIIC04	Snap Impression Coping for Ø4.0	p4
AMIIC05	Snap Impression Coping for Ø5.0	
AMIIC06	Snap Impression Coping for Ø6.0	



EZ PlusTM Internal

Dental Implant Specialty Company, **MegaGen!**



Fixture



Instrument



Abutment



Ref. Code



Memo



Customer Service

[HEAD OFFICE] www.imegagen.com

MEGAGEN IMPLANT CO.,LTD

472, HANJANGGUN-RO, JAIN-MYUN, GYEONGSAN, GYEONGBUK, KOREA, 712-852
5F MegaGen Tower, 607 Seolleung-ro, Gangnam-gu, Seoul, Korea
Tel: 82-53-857-5770
Fax: 82-53-857-5432
E-mail: sales@imegagen.com

[Megagen USA INC.] / USA

440 Sylvan Ave. Suite 200 Englewood Cliffs, NJ 07632, USA
Tel : (+1)866-277-5662
E-mail : info@megagenus.com

[ISG Dental GmbH] / Austria

Wiegelestrasse 10, 1230 Vienna, AUSTRIA
Tel : (+43)1869-3220
E-mail : r.schultschik@isg-gmbh.at

[Implant2dentist] / Denmark

Tandlaegerne i kogade I/S Kogade 4 6270 Toender Denmark
Tel : (+45)7010-8700, (+45)2835-5459
E-mail : mikkelro@implant2dentist.dk

[Megagen Hellas LTD] / Greece

19th G. Papandreou Str., 15773 Zografou, Athens, GREECE
Tel : (+30)21-0748-5533, (+30)21-0748-5544
E-mail : george.kouratiadis@dentomedica.gr

[MegaGen Italia S.R.L.] / Italy

VIA MAZZINI 43/b - 22030 Pusiano (Co), ITALY
Tel : (+39)34-8085-5432
E-mail : info@megagenitalia.it

[UAB "ImplaMedica"] / Lithuania

Kalvariju 272, LT-08316 Vilnius, Republic of Lithuania
Tel : (+37)5219-5971
E-mail : info@implamedica.lt

[MegaGen Benelux b.v.] / Netherlands

Prof. Cobbenhagenlaan 99, 5037 DB Tilburg, The Netherlands
Tel : (+31)8-8848-4100, (+31)6-2000-9582
E-mail : info@megagenbenelux.nl

[Megagen Portugal Lda] / Portugal

Rua dos Açores 67, 4430-643 Vila Nova de Gaia, Portugal
Tel : (+351)2-2488-4026
E-mail : geral@megagen.pt

[MEGAGEN DENTAL IMPLANT] / Romania

STR. Mamulari nr.4, BIC2, sc.2, ap.37 sec.3, Bucuresti
Tel : (+40)3-1418-1238
E-mail : ciprian.amarandei@megagen.ro

[Createch Medical SL] / Spain

Pol. Kurutz Gain, Pab 3 20850 Mendaro (Guipúzcoa), Spain
Tel : (+34)9-4375-7172
E-mail : createch@createchmedical.com

[Orexa Dental AB] / Sweden

Berggrensgatan 8, 412 73 Gothenburg, Sweden
Tel: (+46)709-208000
E-mail : info@orexa.se

[Megagen Implants UK] / United Kingdom

Hemel Business Centre Westside, London Road Apsley Hemel Hempstead Herts HP3 9TD UK
Tel : (+44)14-4284-0326
E-mail : dima.corduneanu@megagen.co.uk

[Abtin Pezeshk Abzar Co.] / Iran

Unit 12, 3th Floor, building pezashkan sabz 2 (No8), Imam zadeh Alley, Opposite Dowlat St,Shariati,Tehran-Iran
Tel : (+98)21-26600568-70 / Fax : (+98)21-22612438
E-mail : apaco2010@ymail.com

[MANDALE HEALTHCARE] / Iraq

Kurdistan region, Sulimanya city, end of Goran street, Salim Bag Building, floor 4, Flat No.1
Tel : (+09)6477-0146-6409
E-mail : yankialawi@yahoo.com

[IMPLAMEDICA] / Lebanon

Implamedica est Centre St. Georgers, Adonis, 4th floor, Lebanon
Tel : (+961)7085-5249; (+961)7001-5532
E-mail : info@megagenlebanon.com

[Dental Guru] / Russia

Bld 43/2, Pokrovka Str. Moscow, 105062, Russia
Tel : (+7)92-6526-2697
E-mail : implantguru@gmail.com

[SAMD Company] / Saudi Arabia

P.O. Box : 381182 Riyadh. 11345, Kingdom of Saudi Arabia
Tel : (+96)6-1431-1188
E-mail : samdcomp@gmail.com

[MegaGen Turkiye] / Turkey

Gulbahar Mah. Buyukdere Cad. No:99 D:7, Mecidiyekoy, Istanbul Turkey
Tel : (+90)21-2211-3932
E-mail : info@megagen.com.tr

[Ningbo Megagen China] / China

No.456 Tonghui road, Jiangbei Inverstment&Pioneering (Park C), Ningbo
Tel : (+86)574-2770-9928
E-mail : megagen@hotmail.com

[MegaGen Hong Kong Ltd] / Hong Kong

Room 604D, InnoCentre 72 Tat Chee Avenue, Kowloon Tong Kowloon, Hong Kong
Tel : (+852)2866-0199
E-mail : megagenhk@gmail.com

[SREE Implantology Solutions India Private limited] / India

D.No.48-12-11B, F.No.201, CBR Residency, Eluru Road, Gunadala, BesideTelephoneExchange,
Vijayawada-520005, AndhraPradesh, INDIA
Tel : (+91)99-6623-8456
E-mail : info@sreeimplants.com

[Dental Solutions & Services Pte Ltd] / Singapore

Blk 28 kallang place #05-16 Singapore 339158 / www.dentalsolutions.sg
Tel : (+65)9147-4633
E-mail : andy.poh@dentalsolutions.sg

[Sunriser Medical] / Taiwan

5 SEC., CHUNG-HSIAO E.RD NANGANG DISTRICT., TAIPEI, TAIWAN 115
Tel : (+88)6-2-27881335
E-mail : sunrise.medical@msa.hinet.net

[Dental-Siam Enterprise Co., Ltd] / Thailand

54 Soi Phanit-Anan, Sukhumvit 71 Road., Klongton Nua, Wattana, BKK 10110 Thailand
Tel : (+66)2713-3355 Ext. 300
E-mail : Praphaporn@dental-siam.co.th

